EN

Diagnosit® BVBLUE® CONTROL KIT Catalog Number S751

INTENDED USE:

The BVBLUE Control Kit is intended for *in vitro* diagnostic use with the BVBLUE Test as unassayed control material to monitor test performance.

SUMMARY:

The controls are for use with the BVBLUE test. The BVBLUE Positive Control is a solution of sialidase that is supplied in a dropper bottle and will produce a green or blue color when used with the test. The BVBLUE Negative Control is a solution of sialidase at a level below the test's limit of detection. It is supplied in a dropper bottle and will produce a yellow color when used with the test. These controls may be used to confirm the integrity of the reagents in the Test Vessel.

REAGENTS:

BVBLUE Positive Control is a solution of ≥ 250 U/mL of bacterial sialidase.
BVBLUE Negative Control is a solution of 50-87.5 U/mL of bacterial sialidase.

Materials Provided:

- 1 bottle (5mL) BVBLUE Positive Control (≥ 10 U/40 μ L)
- 1 bottle (5mL) BVBLUE Negative Control (2-3.5 U/40 μ L)
- Directional Insert

Materials Required but Not Provided:

- BVBLUE Test Kit and Instructions for Use
- Timer or watch
- Swabs

WARNINGS AND PRECAUTIONS:

- For *in vitro* diagnostic use
- Do not use control beyond the expiration date on the label.
- Use disposable gloves while performing the test.
- Used tests, swabs, and other associated materials should be discarded into appropriate waste containers.
- Allow controls to come to room temperature before use.
- Use the amount of control as described in the Instructions for Use (i.e. one drop). Using an excess amount may result in a false result.

STORAGE AND STABILITY:

Store the BVBLUE Control Kit refrigerated at 2-8°C (36-46°F). Do not freeze. The kit is stable until the expiration date shown on the label.

LIMITATIONS OF THE PROCEDURE:

Do not use control beyond the expiration date on the label.

INSTRUCTIONS FOR USE:

Remove the BVBLUE Control Kit from the refrigerator prior to use. Allow the BVBLUE Control Kit to warm to room temperature before use.

If the BVBLUE Test Kit is stored in a refrigerator, remove it from the refrigerator prior to use. Allow the BVBLUE Test Kit to warm to room temperature before use.

POSITIVE CONTROL

STEP 1

Remove one BV Test Vessel and the Developer Solution from the BVBLUE Test Kit prior to use. Remove the cap from the BV Test Vessel.

STEP 2

Add one drop of BVBLUE Positive Control to the BV Test Vessel.

STEP 3

Put the head of a swab into the solution in the BV Test Vessel. Let the BV Test Vessel containing the swab stand for **10 minutes** between 17 and 37°C (62.6 and 98.6°F).

STEP 4

Add one drop of Developer Solution to the BV Test Vessel containing the swab. Gently swirl the mixture.

STEP 5

Read the results immediately after the addition of Developer Solution. See EXPECTED RESULTS Section.

NEGATIVE CONTROL

STEP 1

Remove one BV Test Vessel and the Developer Solution from the BVBLUE Test Kit prior to use. Remove the cap from the BV Test Vessel.

STEP 2

Add one drop of BVBLUE Negative Control to the BV Test Vessel.

STEP 3

Put the head of a swab into the solution in the BV Test Vessel. Let the BV Test Vessel containing the swab stand for **10 minutes** between 17 and 37°C (62.6 and 98.6°F).

STEP 4

Add one drop of Developer Solution to the BV Test Vessel containing the swab. Gently swirl the mixture.

STEP 5

Read the results immediately after the addition of Developer Solution. See EXPECTED RESULTS Section.

EXPECTED RESULTS:

a) **Positive Control Result:** The positive control will produce a blue or green color in the testing vessel and/or on the head of the swab.

b) **Negative Control Result:** The negative control will produce a yellow color in the testing vessel and on the head of the swab.

Note: In the case of weak positive results, take the swab out of the vessel to determine if a blue or green color is present on the swab.

Do not report patient results if either of the two quality control tests does not produce the expected results. If QC testing fails:

- Check the expiration dates of the BVBLUE Test Kit and BVBLUE Control Kit.
- Ensure the instructions for testing were followed.
- Repeat the test.

If the controls still do not perform as expected, contact Gryphus Diagnostics Technical Support at 1-865-251-0101 or customerservice@gryphus.com.

REFERENCES:

Read the directional insert for the BVBLUE Test Kit for more information.

ASSISTANCE:

For technical assistance, call Gryphus Diagnostics Technical Support at 1-865-251-0101.

RE-ORDER:


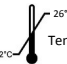
No. BV825E Diagnosit® BVBLUE® Test (25 Tests)
No. S751 Diagnosit® BVBLUE® Control Kit

Manufacturer:

Gryphus Diagnostics, LLC
Knoxville, TN 37919 USA
Tel: 1-865-251-0101

Authorised Representative in the EC:
Winckels Medical Devices Expertise
Bergenweg 18
6085 AT Horn
The Netherlands, Europe
www.wmde.nl

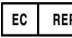Definitions for Symbols

 This product fulfills the requirements of the European Directive for In Vitro Diagnostic Medical Devices	 Batch code	 Catalog number
 Use by	 Authorised representative in the EC	 Contains sufficient for <n> tests
 Manufacturer	 In Vitro Diagnostic Medical Device	 Temperature limitation
		 Consult instructions for use

Diagnosit® BVBLUE® is a registered trademark of Gryphus Diagnostics, LLC.

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Diagnostiik® BVBLUE® on rekisteröity tavaramerkki Gryphus Diagnostics, LLC.